


Put patient into bed


Optional confirmation that patient lies in bed.

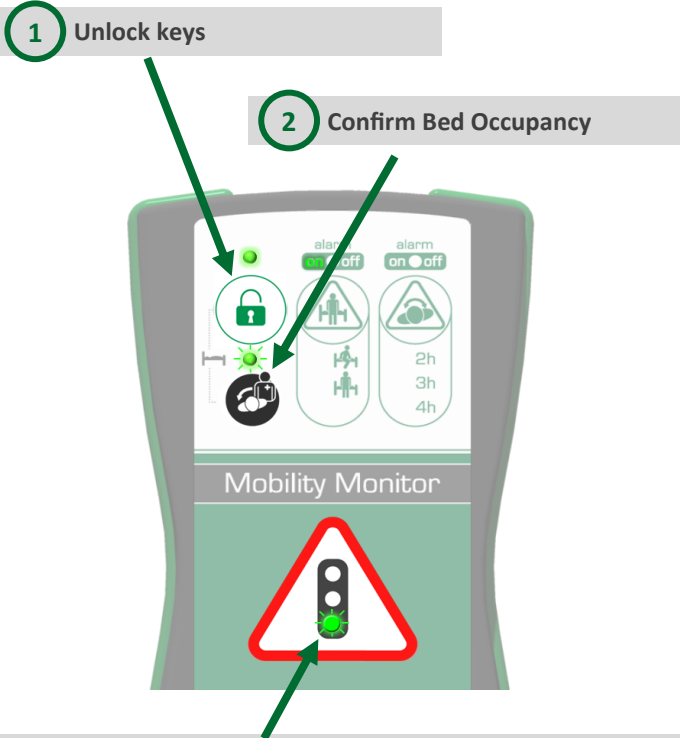

3 The green traffic light stops flashing and is continuously on: The bed exit monitoring is now running (provided that the alert is activated).

Put patient into bed

Optional confirmation that patient lies in bed.

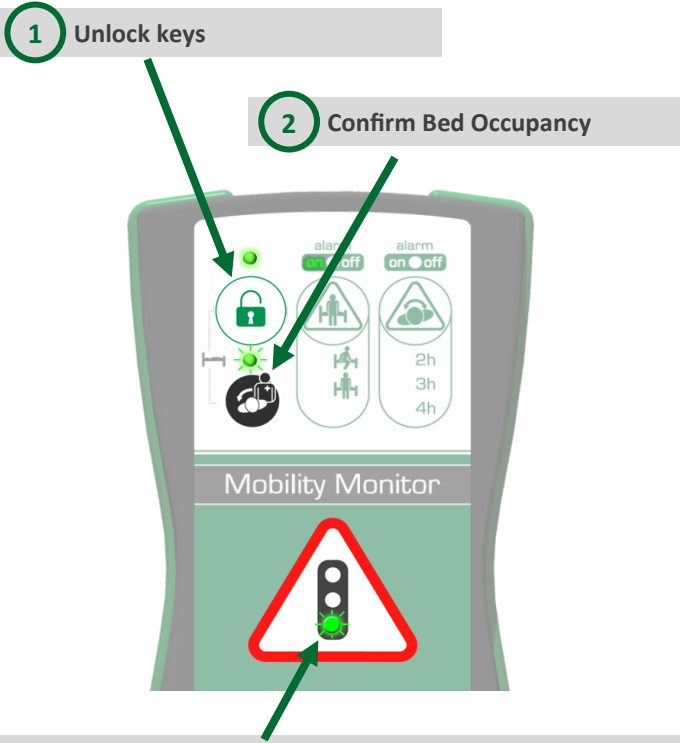


3 The green traffic light stops flashing and is continuously on: The bed exit monitoring is now running (provided that the alert is activated).

Change settings

activate / deactivate alerts

change settings

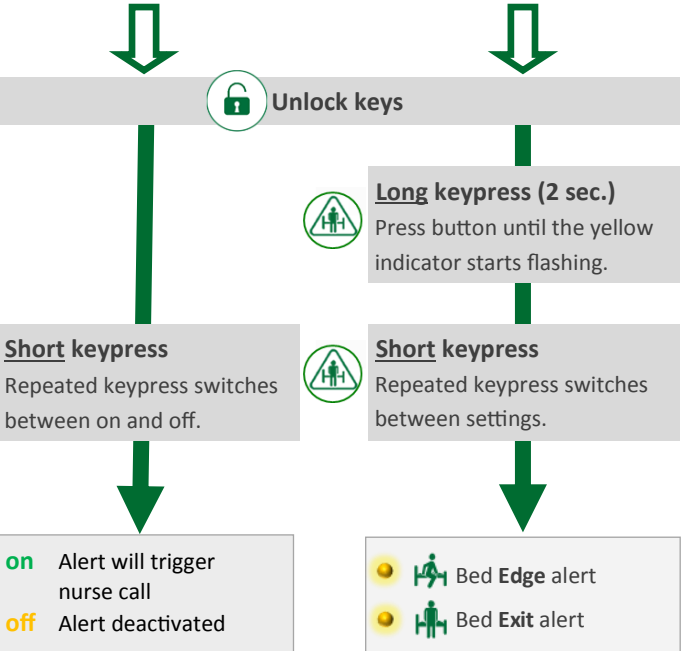


MobilityMonitor_QuickGuide_NightWatch-BedExit_2v2_EN

Change settings

activate / deactivate alerts

change settings

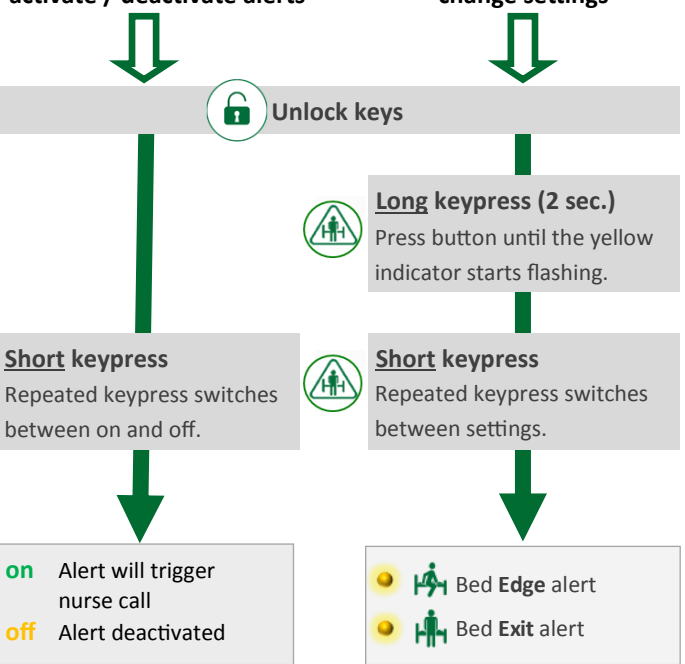


MobilityMonitor_QuickGuide_NightWatch-BedExit_2v2_EN

Assist patient in getting out of bed

Which situation are you facing?

Patient still lies in bed

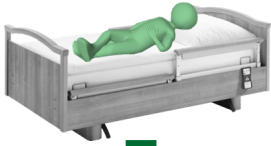

Unlock keys



Patient sitting at the bed edge



Unlock keys



Acknowledge alert



Press the button below the red flashing alert.

alarm off Alert has been triggered
 alarm off Alert has been acknowledged

Deactivate alert



Press button short to deactivate the alert (off). This prevents the device from triggering an alert when the patient gets out of bed.

alarm off Alert is disabled

Patient may now leave the bed

Patient is already out of bed



Unlock keys



Acknowledge alert



Press the button below the red flashing alert.

alarm off Alert has been triggered
 alarm off Alert has been acknowledged

Re-activate the alert



alarm off Alert is activated


MobilityMonitor_QuickGuide_NightWatch-BedExit_2v2_EN



Assist patient in getting out of bed

Which situation are you facing?

Patient still lies in bed

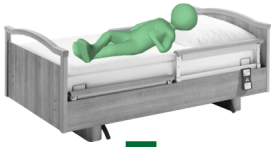

Unlock keys



Patient sitting at the bed edge



Unlock keys



Acknowledge alert



Press the button below the red flashing alert.

alarm off Alert has been triggered
 alarm off Alert has been acknowledged

Deactivate alert



Press button short to deactivate the alert (off). This prevents the device from triggering an alert when the patient gets out of bed.

alarm off Alert is disabled

Patient may now leave the bed

Patient is already out of bed



Unlock keys



Acknowledge alert



Press the button below the red flashing alert.

alarm off Alert has been triggered
 alarm off Alert has been acknowledged

Re-activate the alert



alarm off Alert is activated

MobilityMonitor_QuickGuide_NightWatch-BedExit_2v2_EN

