



Mobility & Care Manager

Overview

Release Date: 08.05.2017

DOC-000019.1 v2.3.0

Table of Contents

Table of Contents	2
Change History	8
1. Introduction.....	3
2. Description Mobility & Care Manager.....	3
2.1. Essential Functions	3
2.2. Licensing	4
3. System Requirements.....	4
4. Application set-up	5
4.1. Local PC / Local Database.....	5
4.2. One or More Local PCs and Centralized Database.....	5
4.3. Terminal Server / Virtualization	6
5. Installation.....	6
5.1. Standard Installation	6
5.2. With MSI Installation Package.....	7
5.3. Comparison of MSI / ClickOnce Installation	7
6. How to Reach Us	8

1. Introduction

This document describes in broad terms the software Mobility & Care Manager of compliant concept AG. For interested customers and IT administrators the main characteristics and the application and installation options are described.

For more details regarding application and functions, please refer to the User's Guide. For more details regarding the installation procedure of the software, please refer to the different installation guides available.

2. Description Mobility & Care Manager

The Mobility & Care Manager software is the application for visualizing and analyzing measurement data of Mobility Monitor by compliant concept. Mobility Monitor data, depending on the model, are transferred via an USB stick or via a radio link into the database of the Mobility & Care Manager software. The database is either installed locally or can be implemented on a central server.

2.1. Essential Functions

- Intuitive operation
- Clear, graphical representation of the mobility data
- Statistical evaluation of the mobility of a patient
- Together with Mobility Monitor Wireless: Real-time display of all devices in use
- Use of a central database supported (MS SQL Server)
- Import data from USB stick (wireless data transfer as option)
- Multiple user handling
- Creating user groups (departments)
- Handling of access rights
- Online Support via integrated TeamViewer (If necessary and upon request remote access to the screen). Can be removed if desired.



2.2. Licensing

After installation, a 60-day trial license is activated. After this period, functionality will be severely limited and a purchased license must be activated.

When the 60-day trial periode has expired, the license must be checked online each time the application is started. A one-time registration per database in use is needed. For the online license check each Mobility & Care Manager instance must be able to connect to our license server over https (<https://licensing.compliant-concept.ch:443>).

3. System Requirements

Operating System	Windows Vista (32 or 64 bit) Windows 7 (32 or 64 bit) Windows 8 (32 or 64 bit) Windows 10 (32 or 64 bit) MS Windows based terminal servers
Database	MS SQL Server 2008R2 or newer MS SQL Express 2012 will be installed during standard installation if not present.
Framework	.Net Framework 4,0 (will be installed if not present)
Processor	1,4 GHz (2 GHz recommended)
Screen Resolution	At least 1024x768
Hard Disk Space	Application: 100 MB SqlExpress: 2,5GB (option) Database: approx. 1GB per year (depending on usage and number of Mobility Monitor devices)
RAM Memory	2 GB (4 GB recommended)
Network Access	Required for access to remote database
Internet Access	Required for - installation with Click-Once setup - automatic activation and registration of licence. - optional for Team Viewer support
USB Port	Required for importing of the mobility data (not applicable for wireless data transfer)
Color Printer	Required for printing nursing records
MSI installation package	Available for download from our website

4. Application set-up

4.1. Local PC / Local Database

In this application, the Mobility & Care Manager software and the database server are installed locally on the PC.

This installation is only recommended if one single person needs to access the data and no central server is available. In this case the required database server will be installed automatically and free of charge.



4.2. One or More Local PCs and Centralized Database

In this application the Mobility & Care Manager software is installed on multiple PCs. The database resides on a central server, so that it is possible to access the same data from different PCs.

This is the recommended installation when multiple users use the Mobility & Care Manager, or if the Mobility Monitor is used in multiple departments.

The central database allows multiple users to access the same data, if the users have permission to view the data for a particular patient or resident. It can be set up that a particular user can only view the patient information for his department. If the patient or resident switches departments, it is possible to transfer the previous data to the next department.



4.3. Terminal Server / Virtualization

The software Mobility & Care Manager can also be used in a virtualized environment. If the software is installed on a Terminal Server, it must be configured in the software settings.

For importing data, the client must be able to recognize a USB stick (mass storage).

Compatible and tested with Citrix XenApp 6.5; other versions are generally possible, but must be tested.



5. Installation

5.1. Standard Installation

The software is provided via Internet link as a ClickOnce deployment. This allows any user (even without admin rights) to install the software themselves. Here the application does not integrate its program files into the Windows program folder, but into each user's profile. Therefore, if multiple users on the same PC need access to Mobility & Care Manager, the installation must be carried out individually by each of these users.

If the data are stored locally, and not stored on a central database server, a standard installation of Microsoft SQL Express is automatically added to the local PC.

Admin rights are only required if basic components (e.g. .NET 4.0) are missing. The user will be prompted for these rights in each case at the appropriate instance. However, administrator privileges are not necessary to execute the software installation file.

5.2. With MSI Installation Package

Upon request we provide IT administrators an MSI package to install the software. This installation option is aimed primarily at larger IT organizations that want to integrate the software into their distribution system.

5.3. Comparison of MSI / ClickOnce Installation

	Standard (ClickOnce)	MSI
Admin rights required	No	Yes
Update notifications	Yes (if internet connection at program startup)	No
Update Time	As soon as published by compliant concept. May be postponed by the user.	Freely selectable.
Required installations	For each Windows user on every PC	Once per PC
Configuration of the software	On the first start-up of the software	Edit MSI or command line parameters for MSI or enter parameters in the installation wizard
Passive installation possible	No	Yes (if server license used)
Installation without TeamViewer	not available	available

6. How to Reach Us

Questions regarding Mobility Monitor and Mobility & Care Manager? We provide technical support via phone or email.

Switzerland (head quarter)

Attendant: +41 44 552 15 00
 Phone Support: +41 44 552 15 03
 Fax: +41 44 552 15 09
 Email: support@compliant-concept.ch

compliant concept AG
 Undermuelistrasse 28
 CH-8320 Fehraltorf
www.compliant-concept.ch

International

Support and Contact Addresses for Distributors
Fehler! Hyperlink-Referenz ungültig. www.compliant-concept.ch/en/Distributors

Change History

Version	Release Date	Change	Prepared	Reviewed	Approved
2.0.0	09.2014	"MobilityAndCareManager_Information_Übersicht_2v0_DE" translated to English	vre	men/zos	kst
2.2.0	07.11.2016	Updated system requirements	men	jdo (pdx)	kst (pdx)
2.3.0	08.05.2017	Added info about online license check	men	jdo (pdx)	bhu (pdx)